

ESI, Inc.

Eye Surgical Instruments, Inc.

2915 Everest Lane
Plymouth, MN 55447

Phone: 763-473-2533
Fax: 763-208-8105



Prager Shell® Tray

The Prager Tray is a convenient storage container for the Prager shell. It may also be used as a container for aseptic cleaning solution that can be used the shell and ultrasonic probe.

Disconnect the tubing kit from Prager Shell® and discard. The Prager Shell® will not properly fit into the tray unless the tubing kit is removed. Place the shell/probe into the storage/soak tray with the Luer fitting and set-screw facing the notched area of the tray. If used as a cleaning tray for disinfection, fill tray approximately 2/3 with an antiseptic solution such as hydrogen peroxide 7.5% . Do not soak the ultrasound probe for a prolonged period a time. After removing the shell/probe from the tray, shake to dispel any antiseptic solution and let air dry. Flush the shell and probe with sterile saline before use. **Caution: Antiseptic solution is harmful to the cornea and must be completely rinsed from the shell and probe before patient use.** If you have any questions regarding the storage tray, contact ESI, Inc.

ESI, Inc.
2915 Everest Lane N.
Plymouth, MN 55447
(763) 473-2533